

REF 6001

Buffing Pad

Intended Use

The Buffing Pad, designed for use on rigid endoscopes, features a buffing surface that helps reduce debris from the lens.

Warnings


- Use only with rigid endoscopes.
- DO NOT use on very hot surfaces.
- DO NOT use with chemicals or solvents.
- DO NOT make any modifications to pads.
- DO NOT bend scope while buffing

In The Event Of An Incident

Any serious incident occurring in connection with this device must be reported to the manufacturer and the relevant authority of the member state in which the user and/or patient is based.



Domicomed.com/eifu

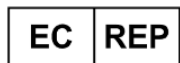

Advena Ltd. Tower Business Centre, 2nd Flr., Tower Street,
Swata, BKR 4013 Malta



Domico Med-Device, LLC
14241 North Fenton Rd.
Fenton, MI 48430
USA
1.800.750.5300
www.domicomed.com

