

REF 6002

Ne-sterilni jastučić za poliranje

Namjena

Ne-sterilni jastučić za poliranje, namijenjen za upotrebu s krutim endoskopima, ima površinu za poliranje koja pomaže ukloniti ostatke s leća.

Upozorenja

- Koristiti samo s krutim endoskopima.
- NEMOJTE koristiti na vrlo vrućim površinama.
- NEMOJTE koristiti s kemikalijama ili otapalima.
- NEMOJTE vršiti nikakve preinake na jastučićima.
- NEMOJTE savijati endoskop tijekom poliranja.

U slučaju incidenta

Svaki ozbiljan incident koji se dogodi u vezi s ovim uređajem mora se prijaviti proizvođaču i nadležnom tijelu države članice u kojoj se nalazi korisnik i/ili pacijent.



Domicomed.com/eif
u



Advena Ltd. Tower Business Centre, 2nd Flr., Tower Street,
Swata, BKR 4013 Malta



Domico Med-Device, LLC
14241 North Fenton Rd.
Fenton, MI 48430
SAD
1.800.750.5300
www.domicomed.com

