

### REF 6002

Non-Sterile Buffing Pad

## Intended Use

The Non-Sterile Buffing Pad, designed for use on rigid endoscopes, features a buffing surface that helps reduce debris from the lens.

## Warnings

- Use only with rigid endoscopes.
- DO NOT use on very hot surfaces.
- DO NOT use with chemicals or solvents.
- DO NOT make any modifications to pads.
- DO NOT bend scope while buffing

## In The Event Of An Incident

Any serious incident occurring in connection with this device must be reported to the manufacturer and the relevant authority of the member state in which the user and/or patient is based.



[Domicomed.com/eifu](http://Domicomed.com/eifu)



Advena Ltd. Tower Business Centre, 2<sup>nd</sup> Flr., Tower Street,  
Swata, BKR 4013 Malta



Domico Med-Device, LLC  
14241 North Fenton Rd.  
Fenton, MI 48430  
USA  
1.800.750.5300  
[www.domicomed.com](http://www.domicomed.com)

